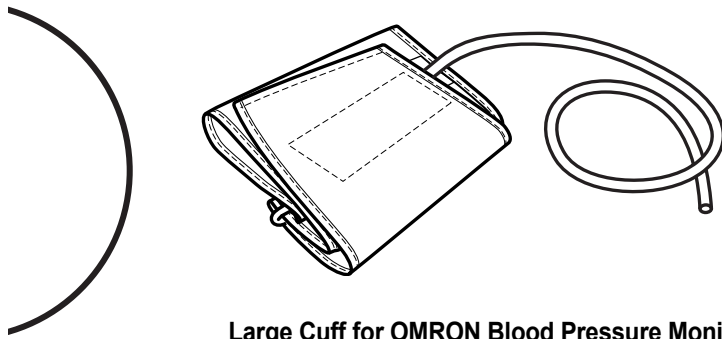
OMRON


Large Cuff for OMRON Blood Pressure Monitors
"MIT Elite Plus" and "MIT Elite"

CL-MIT Elite Large cuff

• <i>Instructions for use</i>	2	EN
• <i>Mode d'emploi</i>	4	FR
• <i>Gebrauchsanweisung</i>	6	DE
• <i>Istruzioni per l'uso</i>	8	IT
• <i>Instrucciones de uso</i>	10	ES
• <i>Gebruiksaanwijzing</i>	12	NL
• <i>Инструкция для пользователя</i>	14	RU

OMRON

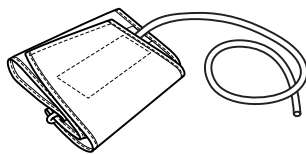
CL-MIT Elite Large cuff

Instructions for use

Thank you for purchasing the OMRON CL-MIT Elite Large Cuff. Arm circumference for this cuff is 32 to 42 centimeters. This Large Cuff can be connected to OMRON "MIT Elite Plus" and "MIT Elite" Upper Arm Blood Pressure Monitor. Please refer to the original instruction manual of your Blood Pressure Monitor.

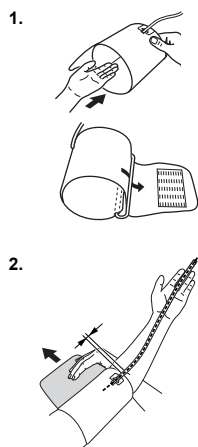Taking measurements on the left arm


1. Put your left arm through the cuff loop.

Note: If the cuff is not assembled, pass the end of the cuff furthest from the tubing through the metal D-ring to form a loop. The smooth cloth should be on the inside of the cuff loop.

2. Position the arm correctly.

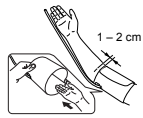- 1) The air tube should run down the inside of your forearm and be in line with your middle finger.
- 2) The bottom of the cuff should be approximately 1 to 2 cm above your elbow.
- 3) Apply the cuff to your upper arm so that the coloured marker (blue arrow under tube) is centered on the middle of your inner arm and points down the inside of the arm.

Taking measurements on the right arm

Apply the cuff so that the air tube is at the side of your elbow.

Notes:

- Be careful not to rest your arm on the air tube, or otherwise restrict the flow of air to the cuff.
- The cuff should be 1 to 2 cm above the elbow.

3. When the cuff is positioned correctly, close the fabric fastener **FIRMLY**.

Hints

- Do not place the cuff over thick clothes and do not roll up your sleeve if it is too tight.
- Align the cuff with the height of your heart (breast height). If necessary use a pillow or cushion.

Maintenance

- Stains on the cuff can be carefully removed using a moist cloth and soapy water.

Do not wash the cuff or make it wet!

- Do not fold the cuff or air tube together too tightly and do not crease them.


This device fulfills the provisions of EC directive 93/42/EEC (Medical Device Directive) as well as clause 4 of EN1060 Non-invasive sphygmomanometers Part 1: General Requirements.

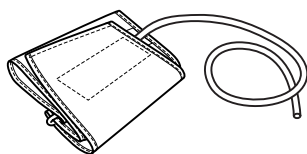Manufacturer 	OMRON HEALTHCARE Co., Ltd. 53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN	EN
EU-representative 	OMRON HEALTHCARE EUROPE B.V. Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS www.omron-healthcare.com	
Production facility	OMRON (DALIAN) CO., LTD. Dalian, CHINA	
Subsidiary	OMRON HEALTHCARE UK LTD. Opal Drive, Fox Milne, Milton Keynes, MK15 0DG, U.K. <hr/> OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH John-Deere-Str. 81a, 68163 Mannheim, GERMANY www.omron-medizintechnik.de <hr/> OMRON SANTÉ FRANCE SAS 14, rue de Lisbonne, 93561 Rosny-sous-Bois Cedex FRANCE	

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